

# Episiotomy Procedural Tray

Catalog # 924/EPL

**The episiotomy procedural kit is a specifically designed kit for episiotomy procedure at childbirth.**

A unique aseptic method for total convenience.

## Including

- 1 Pair disposable complete leg sleeves.
- 4 Sterile drapes.
- 6 Adhesive tapes 2x10cm to attach drapes and leg covers.
- 1 Stainless-steel needle-holder 7" (18cm.) long.
- 1 Europlas™ scissors sharp/blunt.
- 1 Eurofor™ insert forceps.
- 1 Europlas™ curved hemostat.
- 4 Gauze balls X-Ray detectable.
- 10 Gauze sponges 10x10cm X-Ray detectable.
- 1 Gynecological highly absorbent "mouse" tampon.
- 1 Medicine cup 30ml.
- 1 Syringe 20ml.
- 1 Needle 18G.
- 1 Needle 21G.



## Direction for use of kit for Episiotomy procedure at childbirth

### Preparation:

The following additional items are required:

1. 1 pr. surgical gloves of suitable size
2. antiseptic solution
3. sutures & suitable needles
4. anesthetics

### Direction for use:

1. Open the kit, remove the sterile field, & spread over the patient's abdomen or over any other working area with the polybacked surface facing downwards and the absorbent paper surface upwards. It is preferable to secure the sterile field with the elastic plasters to prevent slipping. Use of elastic plaster strips: Gently stretch the plaster lengthways, by using both hands, until the separation is visible at center cut - then easily remove the silicone paper.
2. Hold the tray on its under surface, & empty its contents onto the sterile field, taking care to prevent contact of the edge of the tray with the sterile field. Set the tray aside within easy reach, for use as a container for antiseptic solution during suturing.
3. Fit the disposable leg-sleeves onto the thighs of the patient, securing them with plasters.
4. Spread additional sterile field under the buttocks of the patient, with polybacking facing downwards, - absorbent surface in contact with the patient's body.
5. Additional sterile drape is for use as an apron or any other covering for the patient.
6. Remove the sutures aseptically from their first package, and place them on the sterile field together with the other kit components.
7. Pour antiseptic solution into the kit container. (The compartment is designed for economy)
8. Put on gloves and mask and commence the suturing procedure. Use the gauze & cotton tuffers, the "mouse" tampon, hemostats, syringe plus 2 needles, and medicine cup, as and when required in accordance with your experience in these procedures.
9. During and after the suturing procedure, all refuse must be disposed of in the suitable containers, including all the instruments - all instruments are for single, one time use only - for comfort and safety of patients, personnel and doctors

**Pollak Ltd.** Hamlacha 20, Rosh Ha'ayin, Afek, 48091 Israel.

Tel: 972 3 9022885 Fax: 972 3 9024531 euroband@hotmail.com

[www.euroband.com](http://www.euroband.com)

**EUROBAND**

